

---

**Programming Instructions for the  
Philips HC019C & HC0113C  
and the HC9919C & HC9913C  
Healthcare Television**

*TeleHealth Services  
Healthcare Television Specialsits*

---

# HEALTH CARE (HC) MAIN ON-SCREEN MENU CONTROLS

**I**nstaller: This product is not provided with a pendant control; however, it is expected that this institution (or hospital) will provide one for attachment to the appropriate receptacle on the product. As part of the installation, examine the pendant control to determine whether a marking similar to the one below appears on it:

“CAUTION - Risk of fire if used in oxygen-enriched atmosphere - return to hook or holder after use.”

If the above notice is not provided on the pendant control, remove the label provided with these instructions and attach it to the pendant control.” A separate adhesive-backed label stating “CAUTION” “Risk of fire if pendant control is used in oxygen-enriched atmosphere. Keep pendant control away from oxygen equipment,” can be affixed by the installer to the pendant control if necessary.

Note: Depending on the model and its features, some or all of the displays and features described below will be available. To perform setup operations for the TV Commercial mode, a RG4172BK remote with its TV Setup, Guest, and Data Communication mode capabilities can be used. Press TV SETUP, then press RECALL, to determine whether the TV is in Commercial mode. If the TV is in Consumer mode, press TV SETUP, 02, 49, 95, then M(enu), then use the +/- buttons to select Commercial mode. (See RG4172BK Remote Control Instructions, part number IB7166E002, for details.)

LANGUAGE	ENGLISH/ESPAÑOL/FRANÇAIS	
CHANNEL INSTALL	▶	
CABLE TUNING	ON/OFF	
BRIGHTNESS	█████-----	27
COLOR	█████-----	27
CONTRAST	█████-----	27
SHARPNESS	█████-----	27
TINT	-----+-----	2
NOISE REDUCTION	ON/OFF	
AVL	ON/OFF	
VOLUME BAR	ON/OFF	
MIN VOLUME	█-----	3
MAX VOLUME	█████-----	40
SWITCH ON VOLUME	█████-----	25/STANDARD
CURRENT RING	1/2/3/4	
SWITCH ON CHANNEL	TV Channels 1-125/STANDARD/AUX/S-VIDEO	
POWER ON	STANDARD/FORCED	
CHANNEL DISPLAY	NUMBER/LABEL/ALL/NONE	
KEYBOARD LOCK	ON/OFF	
ESP	1-99/OFF	
AUDIO/VIDEO MUTE	OFF/BLACK/BLUE	
WELCOME MESSAGE	▶	
CHANNEL GUIDE	POWER ON/OFF/ON	
REMINDER	ON/OFF	
3 DIGIT ENTRY	ON/OFF	
A/CH	ON/OFF	
CC	OFF/CC-1/CC-2/CC ON MUTE	
SAVE CC	ON/OFF	
V-CHIP MENU ITEM	ON/OFF	
SAVE V-CHIP	ON/OFF	
V-CHIP SETUP	▶	
SLEEPTIMER	OFF/15/30/45/60/90/120/180/240	
SECURITY	STANDARD/HIGH	
EXIT	▶	

# ON-SCREEN SUB-MENU CONTROLS

## CHANNEL INSTALLATION

CHANNEL	TV 1-125/AUX/S-VIDEO
CHANNEL RING	CHANNEL RING 1/CHANNEL RING 2 /CHANNEL RING 3/CHANNEL RING 4
INPUT	ANTENNA/AUX/S-VIDEO
LABEL	(- - - - -) Example; CHAN 004
VIDEO BLANK	ON/OFF
AUDIO BLANK	ON/OFF
AUTO PROGRAM	▶
EXIT	▶

## WELCOME MESSAGE

MESSAGE	ON/OFF
LINE 1	(- - - - -)
LINE 2	(- - - - -)
CLEAR	▶
EXIT	▶

## V CHIP SETUP

MASTER DISABLE	ON/OFF
TV RATINGS	▶
MOVIE RATINGS	▶
EXIT	▶

## MOVIE RATINGS

G	VIEW/BLOCK
PG	VIEW/BLOCK
PG-13	VIEW/BLOCK
R	VIEW/BLOCK
NC-17	VIEW/BLOCK
X	VIEW/BLOCK
NR	VIEW/BLOCK
EXIT	▶

## TV RATINGS

TV RATINGS	ALL V S L D
TV-Y	[ ]
TV-Y7	[ ] [ ]
TV-G	[ ]
TV-PG	[ ] [ ] [ ] [ ] [ ]
TV-14	[ ] [ ] [ ] [ ] [ ]
TV-MA	[ ] [ ] [ ] [ ]
EXIT	▶

### TO SELECT A FEATURE IN THE ON-SCREEN MENU

Press the POWER button on the remote control. Next, press the TV SETUP button. Then, press the M(enu) button. The TV's on-screen menu will appear. Now, use the ▲ / ▼ buttons to scroll to and highlight the feature you want to adjust. Finally, use the ◀ or ▶ buttons to make the changes. Remember, the TV must be in Commercial mode.

# DESCRIPTION OF MAIN ON-SCREEN MENU CONTROLS

**LANGUAGE** - Highlight LANGUAGE, then press the ◀ or ▶ button to select ENGLISH, ESPANOL, or FRANÇAIS. The Menu and some on-screen displays will now appear in English, Spanish, or French. The language of the Labels and the Channel Guide will not change.

**CHANNEL INSTALL** - Highlight to display the CHANNEL INSTALLATION MENU. This menu contains the following adjustable items: Auto Program, Channel, Channel Ring, Channel Program, Input, Label, Video Blank, Audio Blank and Exit. Refer to CHANNEL INSTALLATION Sub-Menu descriptions.

**BRIGHTNESS, COLOR, CONTRAST, SHARPNESS, TINT** - (Picture Controls) - Highlight the Picture Control you want to adjust, then press the ◀ or ▶ button to adjust the picture control to the desired level.

**NOISE REDUCTION** - Highlight NOISE REDUCTION and press the ▶ button to turn the control ON or OFF. Noise Reduction will help to eliminate signal noise (speckle, snow, etc.) from the TV's picture.

**AVL** - Highlight AVL, then press the ▶ button to turn the control ON or OFF. When ON, AVL will level out the sound being heard, for more consistent listening by reducing the peaks and valleys that occur during program changes or commercial breaks.

**VOLUME BAR** - Highlight VOLUME BAR, then press the ▶ button to select ON. When you press the VOL(ume) ▲ / ▼ buttons, a bar appears on the screen, showing the change in sound. (DISPLAYS must be set to ALL.) Or, press the ◀ button to select OFF. The bar will not appear when you adjust the volume.

**MIN VOLUME** (Minimum Volume) - Highlight MIN VOLUME, then press the ◀ or ▶ button to set the minimum allowable volume.

**MAX VOLUME** (Maximum Volume) - Highlight MAX VOLUME, then press the ◀ or ▶ button to set the maximum allowable volume.

**SWITCH ON VOLUME** - Highlight SWITCH ON VOLUME, then press the ◀ or ▶ button to select a volume level. The volume will be at this level every time you turn on the TV. If STANDARD is selected, the last volume level setting will be used.

**SWITCH ON CHANNEL** - Use this feature to have the TV tune to the same channel every time you turn on the TV. Highlight SWITCH ON CHANNEL, then press the ◀ or ▶ button to select any channel, 1 - 125. (Or, use the Number buttons to select a channel between 1 and 99. Enter a zero before single-digit numbers.) Every time you turn on the TV, the TV will tune to the channel you selected. Or, press the ◀ button to select STANDARD, AUX, or S-VIDEO. When in STANDARD the TV will tune to the channel it was on when it turned off. AUX, or S-VIDEO will tune to those respective connections.

**POWER ON** - Highlight POWER ON, then press the ▶ button to select FORCED. This disables the POWER button on the front of the TV. The TV is now activated when the AC power cord is plugged into an outlet. The SLEEP TIMER feature is disabled on both the remote control and the TV's front panel. With a consumer remote, the TV will not turn off or on. Or, press the ◀ button to select STANDARD. The TV will return to normal operation. (Make sure KEYBOARD is set to ON.)

**CHANNEL DISPLAY** - Highlight CHANNEL DISPLAY, then press the ▶ button to select NONE. This disables any on-screen displays, except SLEEP TIMER and MUTE. Or, press the ◀ button to select ALL. All on-screen displays (such as the Channel Number, Channel Label, or Channel Guide) will reappear. Press the ▶ button again to select NUMBER or LABEL. With NUMBER selected, only the channel number will be displayed. With LABEL selected the display will appear as set in the LABEL control under the CHANNEL INSTALLATION menu.

**KEYBOARD LOCK** - Highlight KEYBOARD LOCK, then press the ▶ button to select ON or OFF. ON disables all the buttons on the front panel of the TV. OFF re-enables the buttons on the front panel of the TV.

**ESP** - This feature allows you to select a time when the TV will automatically turn off. The TV must remain on continuously during the amount of time you select. If the power is switched off, the ESP timer will be reinitialized. Highlight ESP, then press the ◀ or ▶ button to select the number of hours (1 - 99) before the TV turns off. (You also may use the Number buttons to enter the number of hours. For example, press 0, then 6, for six hours. Enter a zero before single-digit numbers.) Or, press the ◀ button to select OFF (or press 0, 0). The TV will not shut off automatically.

**AUDIO/VIDEO MUTE** - Highlight AUDIO/VIDEO MUTE, then press the ◀ or ▶ buttons to select OFF, BLACK, or BLUE. If BLACK or BLUE is selected, the screen will be solid black or blue when the TV is on a channel that is not receiving a signal. The sound will be muted on that channel. Press the ◀ button to select OFF. The normal background static will appear when the TV is on a channel that is not receiving a signal.

**WELCOME MESSAGE** - Highlight WELCOME MESSAGE, then press the ▶ button to display the Welcome Message Sub-Menu. Within this Sub-Menu the WELCOME MESSAGE can be turned ON or OFF. Two lines (LINE 1 and LINE 2) of information can be set to be displayed on the screen when the TV is powered ON. Use the cursor ▲ ▼ buttons to scroll characters for the message lines. Clear will eliminate any message written.

**CHANNEL GUIDE** - Highlight CHANNEL GUIDE, then press the ◀ or ▶ button to select OFF, ON, or POWER ON. If you select POWER ON, the Channel Guide (a list of channels that are labeled) will appear when you turn on the TV. The Channel Guide will appear also when you press the CH GDE (Channel Guide) button. If you select ON, the Channel Guide will appear only when you press the CH GDE button. (Also, if you select POWER ON or ON, make sure CHANNEL DISPLAY is set to ALL.) If you select OFF, the Channel Guide will not appear when you press the CH GDE button. Remember, only the programmed channels that are labeled (within the CHANNEL INSTALLATION MENU) will appear in the Channel Guide.

**REMINDER** - Highlight REMINDER, then press the ▶ button to select ON. The current channel number (and Label, if assigned) will appear on-screen at all times. (Make sure CHANNEL DISPLAY is set to ALL and Closed Captioning is set to OFF.) Or, press the ◀ button to select OFF. The channel number will appear briefly when the channel is selected, then disappear.

**3 DIGIT ENTRY** - Highlight 3 DIGIT ENTRY, then press the ◀ or ▶ button to select ON or OFF. With ON selected the 3 digits can be entered to select channels above 99.

**A/CH** - When highlighted, press the ◀ or ▶ button to select ON or OFF. When ON is selected, pressing the A/CH button will toggle the input source from ANTENNA, AUX, and S-VIDEO. When OFF is selected, pressing the A/CH button will toggle between the currently viewed channel and the last viewed channel.

**CC** - Highlight CC, then press the ▶ button to select the Closed Caption Mode. The choices are OFF, CC-1, CC-2, or CC on MUTE. CC-1 is the most common choice for broadcasters. The CC on MUTE will display the Closed Captioning material when ever the Mute button is pressed.

**SAVE CC** (Save Closed Captioning) - Highlight SAVE CC, then press the ▶ button to select YES. The Closed Captioning mode you selected will be saved, even if you turn off the TV power. Or, press the ◀ button to select NO. Closed Captioning will be turned OFF when you turn off the TV power.

**V-CHIP MENU ITEM** (Censoring Control) - Highlight V-CHIP MENU ITEM, then press the ▶ button to select ON or OFF. When ON, the V-CHIP menu options for the user controls will be shown. When OFF is selected, the user controls will not display the V-CHIP blocking options.

**SAVE V-CHIP** (Censoring Control) - Highlight SAVE V-CHIP, then press the ▶ button to select ON or OFF. ON will save the current V-CHIP settings within the TV's memory.

**V-CHIP SETUP** (Censoring Control) - Highlight V-CHIP SETUP, then press the ▶ button to display the V-CHIP SETUP Sub-Menu. Within this Sub-Menu, certain types of programming maybe set to VIEW or BLOCK. See the V-CHIP SETUP Sub-Menu Descriptions for more details.

**SLEEPTIMER** - Highlight the SLEEPTIMER, then press the ◀ or ▶ button to scroll and select a preset time for the TV to turn itself off. The options are OFF, 15, 30, 45, 60, 90, 120, 180 or 240 minutes.

**SECURITY** - Highlight SECURITY, then press the ◀ or ▶ button to select STANDARD or HIGH. With STANDARD selected, an IR sequence (31, 97, 53, MUTE) will access the SETUP MENU. If HIGH is selected, access to SETUP is only allowed with a Setup Remote.

**EXIT** - Highlight EXIT, then press the ▶ button to remove the on-screen menu from the TV's screen.

# DESCRIPTION OF ON-SCREEN SUB-MENU CONTROLS

## CHANNEL INSTALLATION Sub-Menu

**CHANNEL** - When highlighted, enter the desired channel number using the NUMBERED buttons you wish to edit. The following controls within the CHANNEL INSTALLATION menu will effect the channel chosen.

**CHANNEL RING** - When highlighted, press the ◀ or ▶ buttons to choose CHANNEL RING 1, CHANNEL RING 2, CHANNEL RING 3, or CHANNEL RING 4. Four separate Channel Rings can be setup, then accessed through the main on-screen menu.

**INPUT** - When INPUT is highlighted, press the ◀ or ▶ buttons to select ANTENNA, AUX or S-VIDEO input signals. This control tells the TV to select a specific input when ever this channel is tuned.

**LABEL** - When LABEL is highlighted, press the ▶ button to highlight the first digit area. Then, press the ▲ or ▼ buttons to scroll a list of characters (A-Z, 0-9). Press the ▶ button to highlight the next digit and repeat until the desired label has been entered. When channel changes occur the label chosen will be displayed.

**VIDEO BLANK** - When highlighted, press the ◀ or ▶ buttons to turn the VIDEO BLANK control ON or OFF. When ON, NO VIDEO will be shown when this specific channel is tuned. When OFF is selected, normal viewing will take place.

**AUDIO BLANK** - When highlighted, press the ◀ or ▶ buttons to turn the AUDIO BLANK control ON or OFF. When ON, NO SOUND will be heard when the channel is tuned. When OFF is selected, the channels sound will be normal.

**AUTO PROGRAM** - Highlight CHANNEL INSTALL, then press the ▶ button to display the CHANNEL INSTALLATION Sub-Menu. Highlight AUTO PROGRAM, then press the ▶ button. The TV will automatically search the signal being received for all available channels. Any channels found will be held in the TV's memory and can be accessed using the Channel ▲ or ▼ buttons.

**EXIT** - Highlight EXIT, then press the ▶ button to return to the main menu screen.

## WELCOME MESSAGE Sub-Menu

**MESSAGE** - Highlight MESSAGE, then press the ◀ or ▶ button to toggle the Message control ON or OFF. When ON and a message line has been entered, the Message will be displayed whenever the TV is powered on. The Message will not time out. A press of any button on the remote will remove the Message from the screen.

**LINE 1** - When highlighted, press the ▶ button to highlight the first digit of the line. Use the s or t buttons to scroll a list of available characters (A-Z, 0-9). Press the ▶ button to highlight the next character and repeat the scrolling process until the message has been entered.

**LINE 2** - Use to add a second message or extend the first message. Use the same process as described in LINE 1.

**CLEAR** - When highlighted, press the ▶ button to clear or delete any messages that might be entered in the TV's memory.

**EXIT** - Highlight EXIT, then press the ▶ button to return to the main menu screen.

## V CHIP SETUP Sub-Menu

**MASTER DISABLE** - When highlighted, press the ◀ or ▶ buttons to turn the MASTER DISABLE control ON or OFF. When ON, any blocked programming will be viewable making the V-CHIP censoring inoperable.

**TV RATINGS** - When highlighted, press the ▶ button to display another sub-menu with the TV RATING Options. Press the ◀ or ▶ buttons to set each rating to VIEW or BLOCK.

**TV-Y** - (All children -- This program is designed to be appropriate for all children.) Designed for a very young audience, including children ages 2-6. This type of programming is not expected to frighten younger children.

**TV-Y7** - (Directed to Older Children -- This program is designed for children age 7 and above.) It may be more appropriate for children who have acquired the development skills needed to distinguish between make-believe and reality. This programming may include mild fantasy and comic violence (FV).

**TV-G** - (General Audience -- Most parents would find this program suitable for all ages.) This type of programming contains little or no violence, no strong language, and little or no sexual dialogue or situations.

**TV-PG** - (Parental Guidance Suggested -- This program contains material that parents may find unsuitable for younger children.) This type of programming contains one or more of the following: Moderate violence (V), some sexual situations (S), infrequent coarse language (L), or some suggestive dialogue (D).

**TV-14** - (Parents Strongly Cautioned -- This program contains some material that many parents would find unsuitable for children under 14 years of age.) This type of programming contains one or more of the following: intense violence (V), intense sexual situations (S), strong coarse language (L), or intensely suggestive dialogue (D).

**TV-MA** - (Mature Audience Only -- This program is specifically designed to be viewed by adults and therefore may be unsuitable for children under 17.) This type of programming contains one or more of the following: graphic violence (V), explicit sexual situations (S), or crude indecent language (L).

**MOVIE RATINGS** - When highlighted, press the ▶ button to display another sub-menu with the MOVIE RATING Options. Press the ◀ or ▶ buttons to set each rating to VIEW or BLOCK.

**G: General Audience** - All ages admitted. Most parents would find this program suitable for all ages. This type of programming contains little or no violence, no strong language, and little or no sexual dialogue or situations.

**PG: Parental Guidance Suggested** - This programming contains material that parents may find unsuitable for younger children. It may contain one or more of the following: Moderate violence, some sexual situations, infrequent coarse language, or some suggestive dialogue.

**PG-13: Parents Strongly Cautioned** - This programming contains material that parents may find unsuitable for children under the age of 13. It contains one or more of the following: violence, sexual situations, coarse language, or suggestive dialogue.

**R: Restricted** - This programming is specifically designed for adults. Anyone under the age of 17 should only view this programming with an accompanying parent or adult guardian. It contains one or more of the following: intense violence, intense sexual situations, strong coarse language, or intensely suggestive dialogue.

**NC-17: No one under the age of 17 will be admitted.** - This type of programming should be viewed by adults only. It contains graphic violence, explicit sex, or crude indecent language.

**X: Adults Only** - This type of programming contains one or more of the following: very graphic violence, very graphic and explicit or indecent sexual acts, very coarse and intensely suggestive language.

**NR: Not Rated** - This is programming without rating encoding. This could be such programming as commercials, etc.

**EXIT** - Highlight EXIT, then press the ▶ button to return to the main menu screen.